Water Bottle Kit

**OPERATION
MANUAL**

CE MADE IN JAPAN

English	1
Deutsch.....	29
Français.....	57
Español	85
Italiano.....	113
Português.....	141

TABLE OF CONTENTS

English

1	Preface	3
1.1	User and intended purpose.....	3
1.2	Related manuals	3
1.3	Precautions for handling and operation	3
1.4	Symbol	5
1.5	Disposing of products	6
1.6	Warranty	6
2	Product Description	7
2.1	Package contents.....	7
2.2	Part names	8
3	Preparation for Use.....	9
3.1	Attaching the water bottle module.....	9
3.2	Connecting hose	10
3.3	Connecting irrigation bottle and CLM bottle.....	11
3.3.1	Connecting.....	11
3.3.2	Disconnecting	12
3.3.3	Opening/closing the bottle lid.....	12
3.4	Check before treatment	13
4	Operation Procedure.....	14
5	Post-use Maintenance	15
5.1	Preparation for maintenance.....	15
5.2	Irrigation bottle and water bottle module	16
5.2.1	Irrigation bottle and bottle lid.....	16
5.2.2	Water bottle module and purge tool	16
5.3	Auto-cleaning and irrigation bottle.....	17
5.3.1	Auto-cleaning.....	17
5.3.2	Irrigation Bottle and bottle lid	17
5.4	Decontamination and prevention of biofilm formation of water circuit (twice per week)	18
5.4.1	Preparation.....	18
5.4.2	Preparation of CLM mode.....	18
5.4.3	CLM mode	19
5.4.4	Bottle lid.....	20
5.4.5	CLM Bottle	21
5.4.6	Purge tool	21
5.5	Storage	21
6	Maintenance	23
6.1	Maintaining O-rings	23
6.1.1	Irrigation bottle and CLM bottle.....	23
6.1.2	Purge tool	23
6.2	Air filter.....	24
6.3	Periodical maintenance	24
6.4	Replacing consumable parts	24
7	Troubleshooting	25
7.1	Problems and solutions.....	25

8	Spare Parts and Option List.....	26
9	Specifications.....	27

1 Preface

1.1 User and intended purpose

- User
Dentist, Dental hygienist
- Intended purpose
Water Bottle Kit is intended to supply water in connection with the Water Hose connector of Varios Combi Pro2.

1.2 Related manuals

For proper use with the combined items, refer to the following operation manuals:

Operation Manual Title	Operation Manual No.
Varios Combi Pro2 OPERATION MANUAL	OM-DE1002MA

1.3 Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety indicators are intended to avoid potential hazards that could result in personal injury or damage to the product. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of risk
▲WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
▲CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

▲WARNING

Environment

- Use this product in a location that meets the requirements for the use environment and does not produce condensation. Otherwise, it may cause malfunction. => "9 Specifications".

Designated items

- Be sure to do not use other than accessories of this product included package and NSK's genuine (designated item). The user is liable for accidents and malfunctions that occur as a result of the use of items other than the accessories of this product included in the package and NSK's genuine products (designated items).

EMC

- Do not install the purge tool near electronic equipment. This may affect the operation of the electronic equipment.

WARNING

Installation

- Do not drop, knock, or subject the bottle to excessive shock. Do not use the bottle if it is cracked or damaged, as it may break and cause injury to the patient and operator.

CAUTION

Environment

- Set the supply air pressure to 0.45-0.75 MPa and use clean and dry air. Using with the incorrect pressure may result in insufficient spray pressure (power), inoperability, malfunction or the bottle may explode and cause injury.
- Be sure to use the drinking water with salinity concentration of 0.5 % or less and temperature less than 35 °C. Using other liquids may cause water clogging or metal corrosion.

Designated items

- Only use the hose specified. The use of non NSK hoses or hoses in a deteriorated condition may lead to the following risks.
 - Cannot be connected or fixed.
 - Damage to hoses or connections causing the hose to disconnect.
 - Water leak / air leak
- Only use the bottle specified. The use of non NSK bottles may lead to the following risks.
 - Cannot be connected or fixed.
 - Injuries caused by broken bottles.
 - Water leak

Connections

- Do not forcibly remove hoses. Doing so may cause damage to the hose or parts.

Before use

- Be sure to perform inspection by operating the product with the each connecting part mounted before use. => “3.4 Check before treatment”.
If any abnormal condition is detected, stop using the product and contact your Authorized NSK Dealer.
- If the product has not been used for a long period, check that it is functioning correctly and safely before using on a patient.
- Wear eye protection, a mask, and non-contaminated clean gloves to prevent infection.

During use

- When removing the irrigation bottle or CLM bottle, press the Bottle Release Button before removing the bottle. Forcibly removing the irrigation bottle or CLM bottle may damage the bottle lid connector and bottle joint.

⚠ CAUTION

Maintenance

- Do not attempt to disassemble the product or tamper with the mechanism except as recommended by NSK in this manual.

Storage

- When not using the water bottle module, always insert the purge tool into the bottle joint to prevent ingress of contamination or debris.

1.4 Symbol

Prohibited / Wrong process / Incorrect

Do this / Correct process / Correct

Listen (for a click)

Consult operation instructions

Temperature

Drinking water

Caution

The CE marking certifies that this product has met legal requirements in force in the European Union (EU)

Manufacturer

Importer

Authorized representative in the European Community

Authorized representative for Switzerland

Catalog number (Order code)

Serial No.

Date of manufacture

Use-by date

Contains hazardous substances

Temperature limitation

Humidity limitation

Atmospheric pressure limitation

1.5 Disposing of products

When your device has reached the end of its service life, it must be disposed of via a specialist collection, removal, recycling or destruction channel. Prior to doing so, in order to avoid the health risks to operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, you need to confirm the equipment is decontaminated.

Therefore contact your nearest NSK dealer, or one of the NSK branches to find out how to proceed.

Refer to the link below:

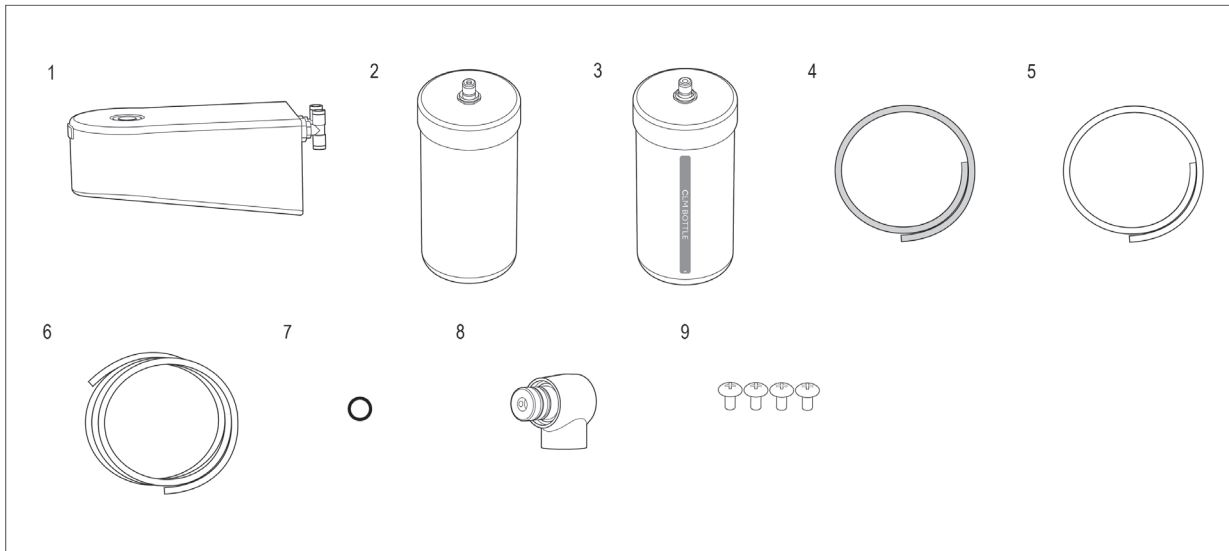
1.6 Warranty

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model. Contact an authorized NSK dealer if repairs are necessary.

2 Product Description


2.1 Package contents


No.	Part Name	Quantity
1	Water Bottle Module	1
2	Irrigation Bottle Set	1
3	CLM Bottle Set	1
4	Water Hose	1
5	Air Hose (0.7m)	1
6	Air Hose (3.6m)	1
7	O-ring	1
8	Purge Tool	1
9	Screws	4

2.2 Part names

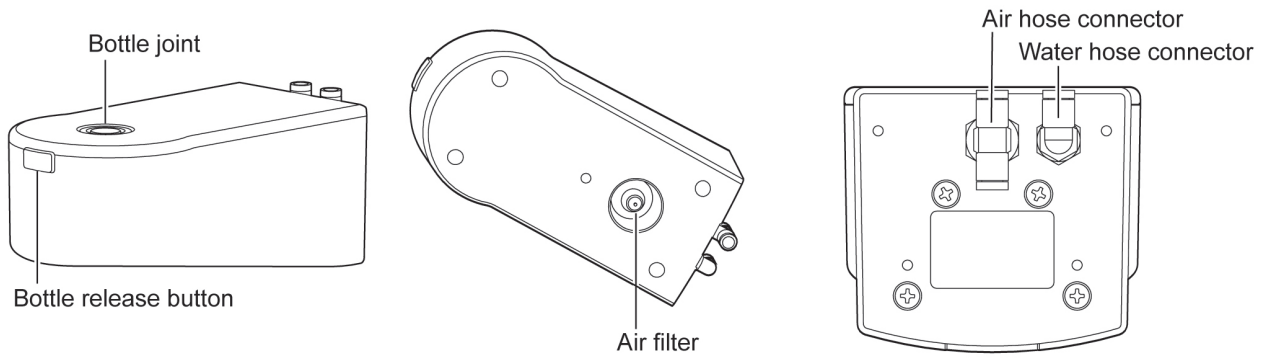
■ Water bottle module

■ Irrigation Bottle

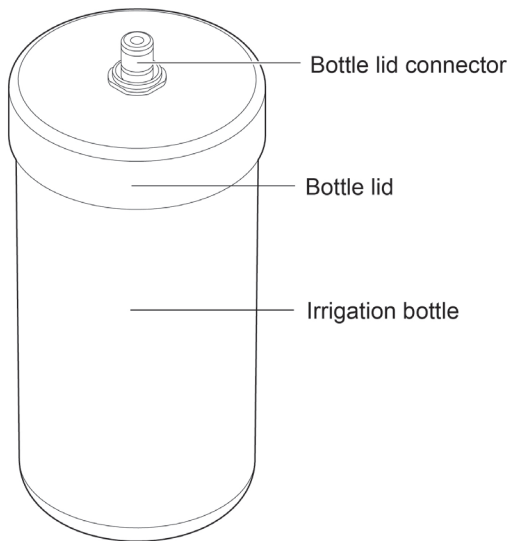
■ CLM Bottle

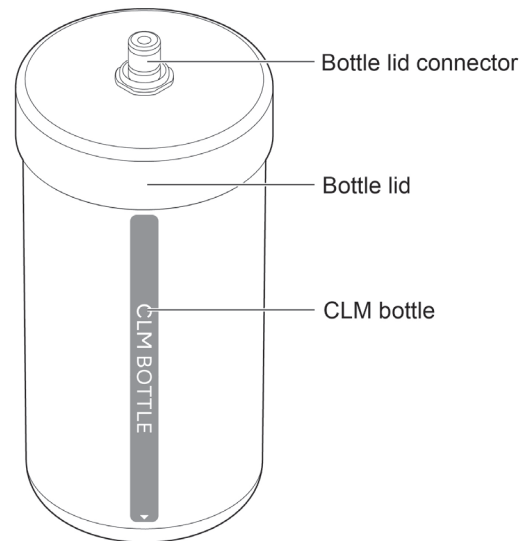
3 Preparation for Use

3.1 Attaching the water bottle module

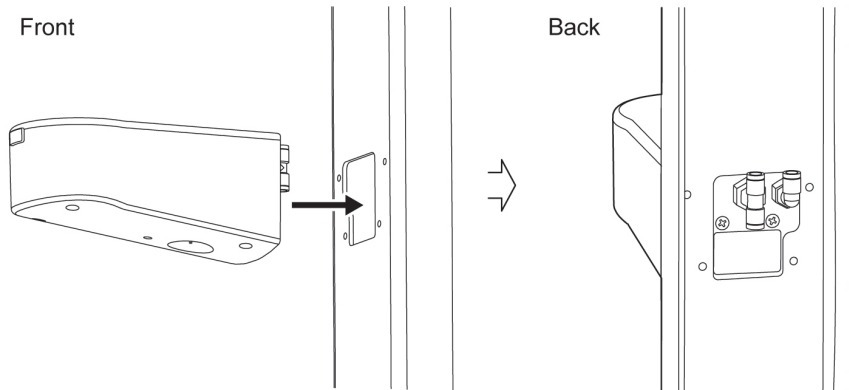1. Pass through the air hose connector and water hose connector of the water module into the passing hole of the cart (prepared by customer).

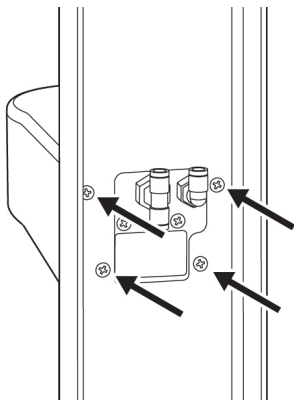2. Align the 4 tapped holes on the water module with the holes on the cart. Fix using screws and a Phillips screwdriver.

⚠ CAUTION

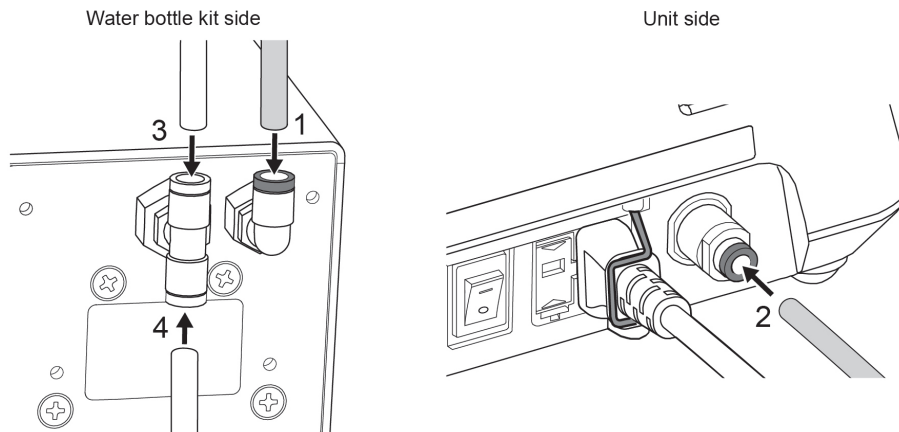
- After attaching the water bottle module, check for rattling. If rattling occurs, tighten the screws.

3.2 Connecting hose

To confirm there is no damage to each of the hoses.

■ Water hose and air hose

1. Insert one end of the water hose into the water hose connector with the green ring on the water module.
2. Insert the water hose into the water hose connector with the green ring on the control unit.
3. Insert the air hose (0.7 m) of the control unit side into the air hose connector (upper side) with the white ring on the water module.
4. Insert the air hose (3.6 m) of the dental chair unit side into the air hose connector (under side) with the white ring on the water module.

5. Pull the water hose and air hose gently, and check that they do not come out.

To remove:

Push and hold the green ring on the water hose connector and pull out the water hose.

Push and hold the white ring on the air hose connector and pull out the air hose.

⚠ CAUTION

- When connecting the water hose to the water hose connector on the water bottle module, insert it firmly all the way in. If it is not inserted firmly, it may result in water leakage.
- Be sure to connect the water hose to the water hose connector with the green ring. If the water hose is connected to the air hose connector with the white ring, water enters the control unit causing a malfunction.
- When removing the water hose, be sure to perform auto-cleaning and remove the water inside the water hose.
- Be sure to connect the air hose to the air hose connector (white ring).
- Do not forcibly remove hoses. Doing so may damage hoses or parts.
- Be sure to confirm that air has stopped before attaching or removing the air hose.

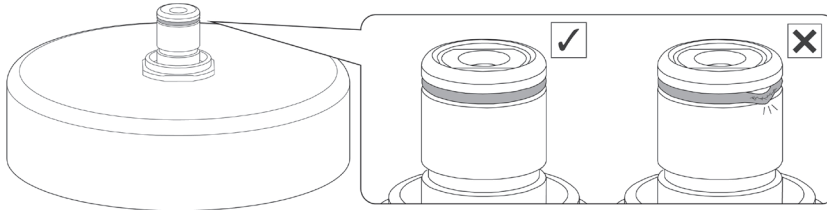
3.3 Connecting irrigation bottle and CLM bottle

In this operation manual, both the irrigation bottle and CLM bottle are referred to as “bottle”.


3.3.1 Connecting

1. Confirm each part:

– Check that the O-ring is attached on the bottle lid connector of the bottle and that it is not damaged.

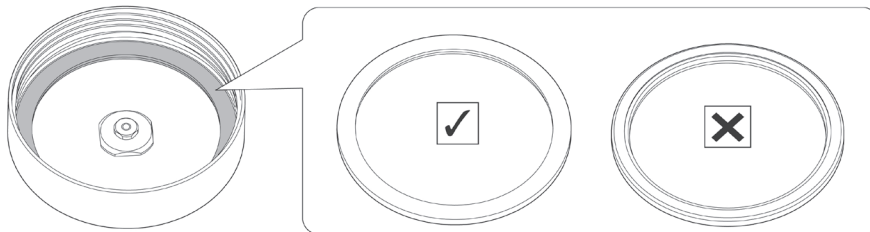
– Confirm that the gasket is properly attached to the bottle lid.


– Check that the bottle and bottle lid is not damaged.

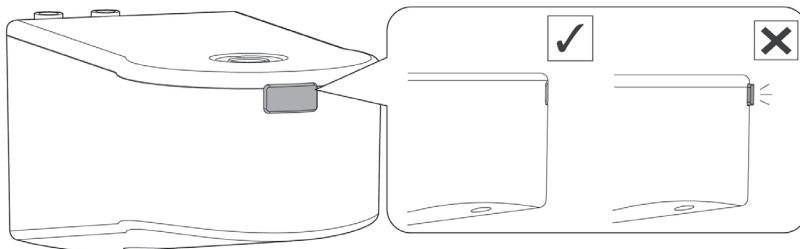– Check that the bottle release button of the water bottle module is as shown in the ✓ illustration below.

If it is as shown in the ✗ illustration, press the bottle release button so that it appears as shown in the ✓ illustration.

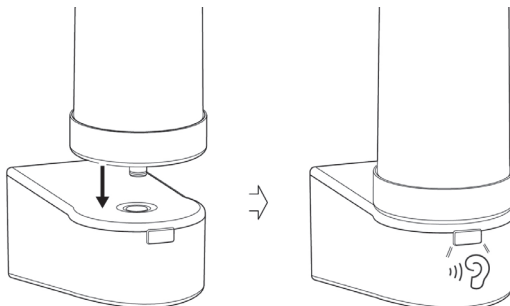2. Fill each bottle with the specified water or disinfectant, tighten the bottle lids, and confirm that there is no leakage.
(Irrigation bottle: Water, CLM bottle: CLM disinfectant)
3. Insert the bottle lid connector of the bottle straight into the bottle connector until it clicks.

※After connecting the bottle, if water leaks from the connection, immediately press the bottle release button and pull out the bottle.

⚠ CAUTION

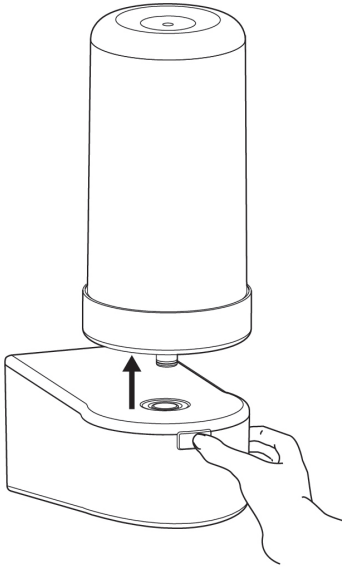
- Inserting the gasket of the bottle lid in the incorrect direction may cause water leakage.

⚠ CAUTION

- Always securely tighten the bottle lid. Failing to do so may cause water to leak from the bottle.
- Be sure to use a genuine NSK gasket. Fitting another gasket may cause water to leak.
- Insert each bottle straight. If the bottle is inserted at an angle it may cause damage to the O-ring.

3.3.2 Disconnecting

While holding the bottle press the bottle release button on the water bottle module to disconnect the bottle.


3.3.3 Opening/closing the bottle lid

⚠ CAUTION

- Before loosening the bottle lid, be sure to use the purge tool to release the compressed air from inside the bottle. If the bottle is forcibly loosened without releasing the pressure from the irrigation bottle, the following may occur due to sudden pressure release.
 - Loud noise is generated.
 - Water splashes.

■ Bottle purging

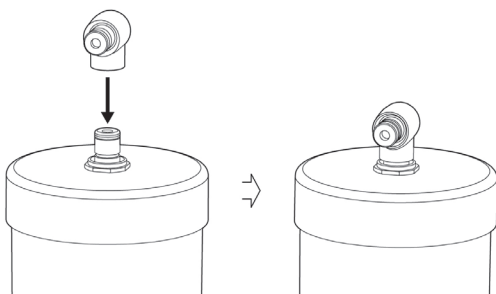Insert the purge tool into the bottle lid connector, then remove the compressed air from inside the bottle by pushing the purge tool.

Press and hold until no air is heard escaping from the tip of the bottle lid connector.

NOTICE

- A pressure of up to 0.2 MPa will be present inside the bottle before purging.
-

3.4 Check before treatment

Before use, operate the product without a patient present and check the following points.

- Water bottle module

Confirm that there is no sound of leaking air.

- Bottle

Confirm that there is no damage, deterioration, or water leakage from the irrigation bottle or CLM bottle and bottle lid.

- Each hose

Confirm there is no damage present on each hose.

- Each connection part

Confirm each connection part is securely connected.

For information on what to inspect before use, refer to “3.15 Check before treatment” in the “OM-DE1002MA Varios Combi Pro2 OPERATION MANUAL” listed in “1.2 Related manuals”.

Confirm that parts have been irrigated.

If any abnormality is detected during the check or use, stop operation immediately and contact your authorized NSK dealer.

4 Operation Procedure

NOTICE

- For information on the operation of the control unit, refer to the Varios Combi Pro2 OPERATION MANUAL.
-

5 Post-use Maintenance

Perform the following maintenance after use on each patient.

The following reprocessing instructions are created on the basis of the requirements of ISO 17664.

The effectiveness of the reprocessing through the following procedure has been verified.

⚠ WARNING

- For cleaning and disinfection, use disinfectant ethanol or disinfectant isopropanol that does not contain any additives (any quaternary ammonium salts). Use of other disinfectants may cause discoloration or cracking.
 - * For details, contact the manufacturer of the disinfectant.
 - * In this operation manual, disinfectant ethanol or disinfectant isopropanol that does not contain any additives (any quaternary ammonium salts) is called "disinfectant alcohol".
- Use appropriate disinfectant alcohol according to the regulations and guidelines of each country.
- Always obey the local directives, standards, guidelines, etc., as to the cleaning, disinfection, and sterilization of the product.
- Use only the cleaning and disinfection agent products specified in this operation manual.
- Do not use solvents such as electrolyzed-oxidizing water (strong acidic water or super acidic water), strong acidic or alkaline chemical, chlorine-containing solvent, benzene and thinner when cleaning, immersing, or wiping.
- Improperly maintaining the product may result in infection, product failure, or user or patient burn due to reduced irrigation volume.
 - Be sure to clean and disinfect the products immediately (within 30 minutes) after use to remove residue.
- The product lifetime and ability to operate correctly are mainly determined by mechanical stresses during use. If you detect any abnormality such as abnormal vibration, action or noise, or if the exterior markings (serial number or model number) have become unreadable before or during use, stop using the product immediately and request your nearest NSK dealer, or NSK branch for a periodic inspection*.
 - Disappearance of exterior markings is not covered under warranty for repair.

*Refer to the link below:

5.1 Preparation for maintenance

Wear eye protection, a mask and uncontaminated clean gloves to prevent infection.

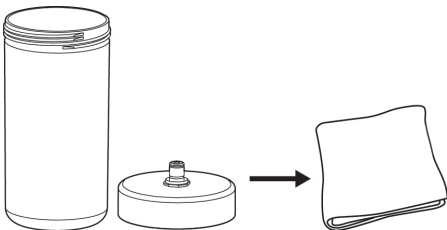
5.2 Irrigation bottle and water bottle module


5.2.1 Irrigation bottle and bottle lid

■ Cleaning


1. Wipe off soil from the surfaces of the irrigation bottle and bottle lid using a lint-free cloth dampened with water.

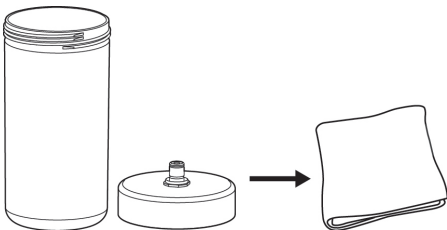2. Wipe the surfaces of the irrigation bottle and bottle lid using a lint-free cloth dampened with disinfectant alcohol.

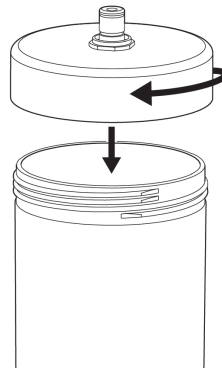3. Once they are dry, tighten the bottle cap on the water-filling bottle.


4. Proceed to storage. => "5.5 Storage".


5.2.2 Water bottle module and purge tool

■ Cleaning

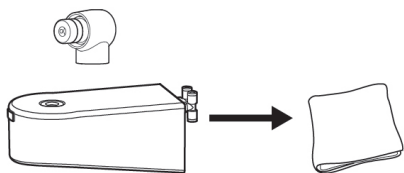
1. Wipe off soil from the surfaces of the water bottle module and purge tool using a lint-free cloth dampened with water.

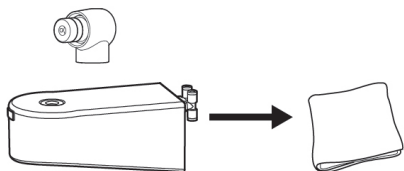2. Wipe the surfaces of the water bottle module and purge tool using a lint-free cloth dampened with disinfectant alcohol.

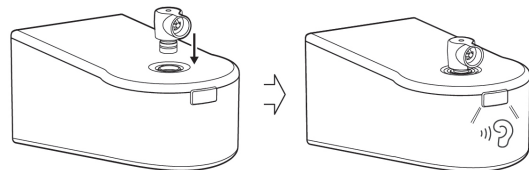3. Insert the purge tool straight into the bottle connector of the water bottle module until it clicks.


4. Proceed to storage. => "5.5 Storage".

5.3 Auto-cleaning and irrigation bottle

Perform the following steps after the procedure or as needed.

5.3.1 Auto-cleaning

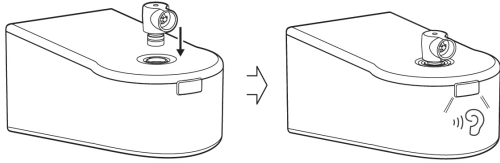1. Insert the purge tool into the bottle joint of the water bottle module.


2. Perform auto-cleaning.
For auto-cleaning, refer to “5.2 Auto-cleaning”, and “7.3.3 Choosing the correct cleaning mode program” of the OM-DE1002MA Varios Combi Pro2 OPERATION MANUAL listed in “1.2 Related manuals”.
3. After auto-cleaning is complete, while holding the purge tool by hand, press the bottle release button to disconnect the purge tool.


5.3.2 Irrigation Bottle and bottle lid

■ Rinsing and disinfection

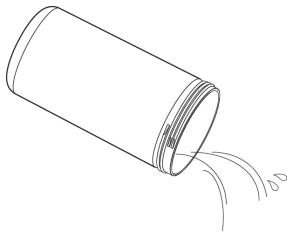
1. Drain the water remaining in the irrigation bottle.

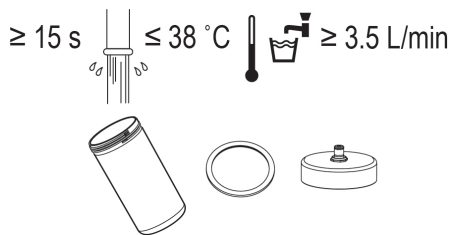
2. Rinse the irrigation bottle, bottle lid and gasket using running water for 15 seconds.

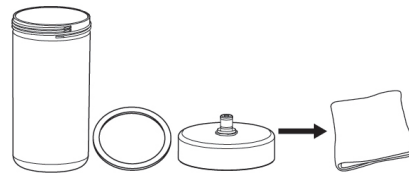3. Use a (lint free) cloth moistened with disinfectant alcohol to wipe the inside and outside of the irrigation bottle, and the outside of the irrigation bottle lid.

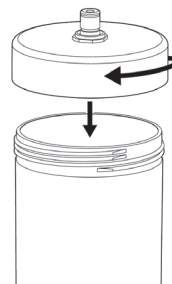4. Tighten the bottle lid to the irrigation bottle.

5. Proceed to the storage. => “5.5 Storage”.

⚠ WARNING

- Be sure to perform maintenance of the water circuit at the end of the working day. If proper maintenance is not performed, it may cause infection or clogging.

5.4 Decontamination and prevention of biofilm formation of water circuit (twice per week)

Disinfect the water circuit using b.clean or Bilpron (ALPRO) (undiluted).

Be sure to perform auto-cleaning before the maintenance of the water circuit.

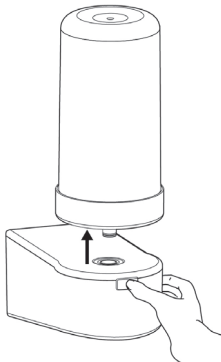
In this chapter, maintenance of water circuit is collectively referred to as CLM Mode.

⚠WARNING

- Disinfect the water circuit using b.clean or Bilpron (ALPRO) (undiluted).
* In this operation manual, b.clean or Bilpron (ALPRO) is called "disinfectant for CLM".
- Be sure to perform maintenance of the water circuit twice a week at the end of the working day. If proper maintenance is not performed it may cause infection or clogging.
- Because chemicals are used in CLM mode, the product cannot be used for usual treatment until CLM mode is complete.
- For the maintenance of this product, use the detergent and disinfectant specified in this operation manual. The use of detergent or disinfectant other than those specified in this manual may cause a failure of this product, maintenance failure, and/or the patient's infection.



5.4.1 Preparation

1. Remove the irrigation bottle from the bottle joint.
2. Disinfect the irrigation bottle. => "5.3.2 Irrigation Bottle and bottle lid" and "5.4.4 Bottle lid".

5.4.2 Preparation of CLM mode

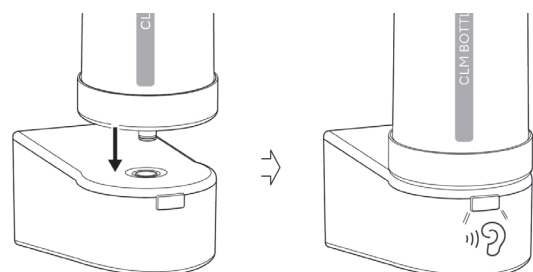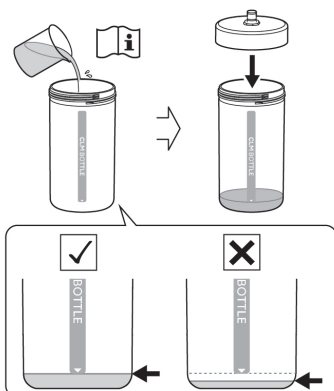1. Fill the CLM bottle with b.clean or Bilpron (ALPRO) (undiluted), and then tighten the bottle lid. Check to ensure there are no leaks.
2. Insert the CLM Bottle straight into the bottle connector.

5.4.3 CLM mode

■ Filling

1. Fill with disinfectant solution.
For CLM mode, refer to “5.10.1 Preparation” of the OM-DE1002MA Varios Combi Pro2 OPERATION MANUAL listed in “1.2 Related manuals”.
2. After the CLM water circuit is filled with disinfectant, the following screen is displayed on the control unit and  flashes.

3. Press the bottle release button and remove the CLM bottle.

4. Insert the purge tool into the bottle lid connector of the CLM bottle, and then remove the compressed air from inside the bottle while pressing the purge tool. Continue to press until you no longer hear the sound of air being released from the bottle lid connector.
5. Rinse the CLM Bottle. => “5.4.5 CLM Bottle”.
6. Insert the purge tool into the water bottle module.
7. Turn off the power to the control unit, and then leave the product alone for 12 hours or more for disinfection.

⚠ CAUTION

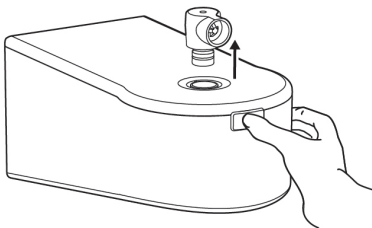
- After filling with disinfectant for CLM, be sure to remove the CLM bottle and remove air from the inside of the bottle. Leaving the bottle with pressure remaining inside may result in premature failure.

■ Rinsing


1. After 12 hours, turn on the power, a beep is heard, following screen is displayed on the control unit, and  flashes.


2. Remove and rinse the purge tool. => “5.4.6 Purge tool”

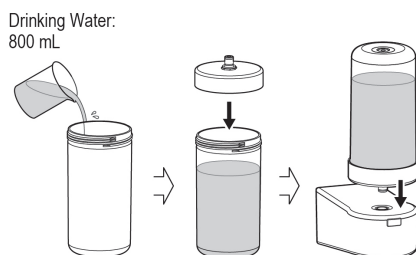3. Fill the disinfected irrigation bottle with 800 mL of drinking water, tighten the bottle lid, and then insert the irrigation bottle straight into the bottle connector.

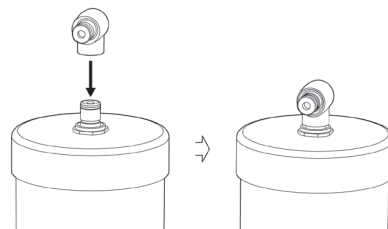4. Press the CLM key of the control unit for three seconds. For CLM mode, refer to “5.10.2 CLM mode” of the OM-DE1002MA Varios Combi Pro2 OPERATION MANUAL listed in “1.2 Related manuals”
5. Once the CLM mode has been completed, remove the irrigation bottle from the bottle joint. => “3.3.2 Disconnecting”.
Insert the purge tool into the irrigation bottle and then purge it.
* Should the unit not be used to treat patients proceed to storage. => “5.5 Storage”.


6. Switch the operation mode of the control unit to powder mode, and then depress the operation mode/rinse button of the foot control for 15 seconds to drain water from inside the module.
7. Perform check before treatment before starting the treatment. => “3.4 Check before treatment”.


5.4.4 Bottle lid

■ Disinfection

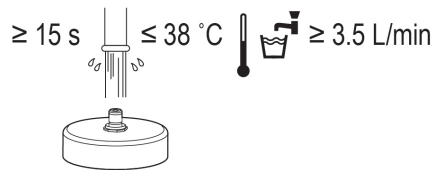
1. Rinse the bottle lid using running water.

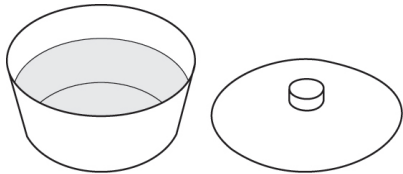2. Prepare disinfectant alcohol in container.

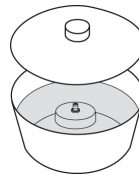
3. Soak the bottle lid in the disinfectant alcohol.


4. Make sure that no bubbles form on the bottle lid. To prevent the disinfectant from vaporizing, seal the container and let it sit for six hours.

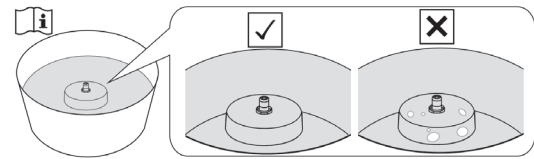* If there are any bubbles, remove them.


■ Rinsing

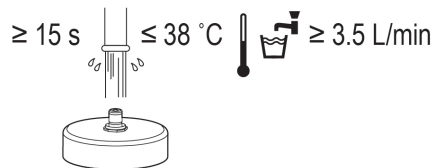1. Take the bottle lid out of the disinfectant, and then rinse using drinking water.

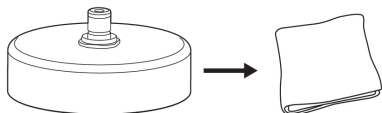2. Wipe the water off the surfaces of the bottle lid using a clean and dry cloth.


3. Tighten the bottle lid on to the irrigation bottle.
4. Proceed to storage. => "5.5 Storage".


5.4.5 CLM Bottle

■ Rinsing

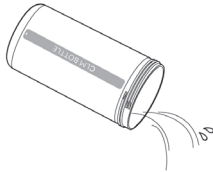
1. Drain the disinfectant for CLM from the CLM bottle.

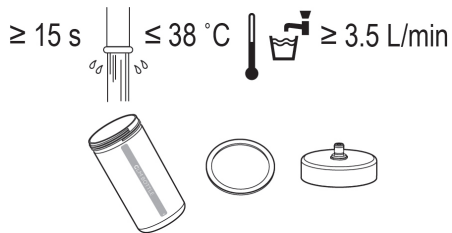
2. Remove the bottle lid from the CLM bottle.
3. Remove the gasket from the bottle lid.
4. Rinse the CLM bottle, bottle lid, and gasket using running water.

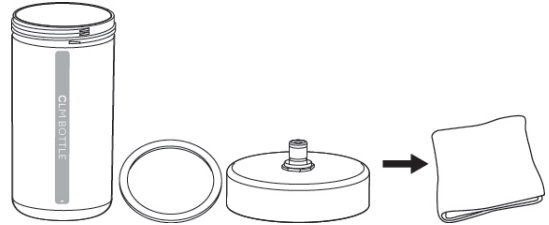5. Wipe the water off the surfaces of the CLM bottle, bottle lid, and gasket using a clean and dry cloth.

6. Attach the gasket to the bottle lid.
7. Tighten the bottle lid to the CLM bottle.

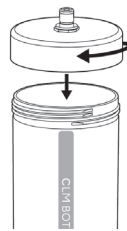
8. Proceed to storage. => "5.5 Storage".


5.4.6 Purge tool

■ Rinsing

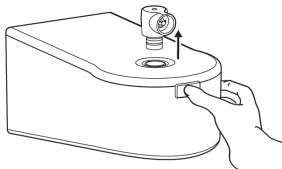
1. Press the bottle release button and remove the purge tool.

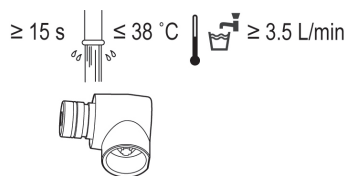2. Rinse the purge tool using running water.

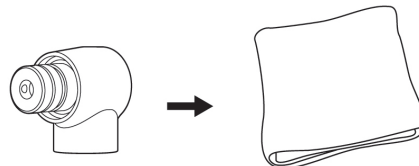3. Wipe the surfaces of the purge tool using a lint-free cloth dampened with disinfectant alcohol.

4. Proceed to storage. => "5.5 Storage".

5.5 Storage

Before storing the water bottle module, perform auto-cleaning and then drain water from the water circuit. After performing auto-cleaning, attach the purge tool and then store the water bottle module.

For auto-cleaning, refer to the OM-DE1002MA Varios Combi Pro2 OPERATION MANUAL listed in "1.2 Related manuals".

Store in a clean environment when not in use.

CAUTION

- Be sure to perform auto-cleaning to drain water from the water circuit before storage.
Store the product in a well ventilated place out of direct sunlight and within the range of temperature, humidity and pressure specified in “9 Specifications”. The air should be free from dust, salt and sulphur.

6 Maintenance

6.1 Maintaining O-rings

Perform O-ring maintenance to prevent O-ring wear and water leakage in each area.

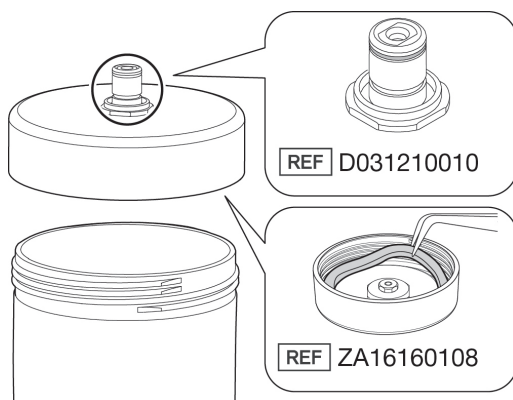
6.1.1 Irrigation bottle and CLM bottle

- Bottle lid connector

1. Remove the O-rings for the bottle from the bottle lid connector using a needle or similar tool.
2. Insert new O-rings into the grooves.


- Bottle lid

1. Remove the gasket from the bottle lid.
2. Attach the new one to the bottle lid.

6.1.2 Purge tool

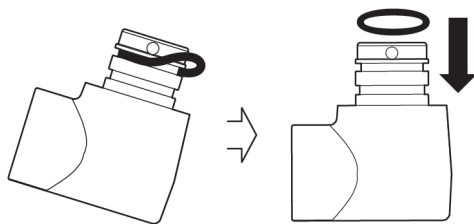1. Remove the O-ring from the purge tool.
2. Attach the new one to the purge tool.


⚠ CAUTION

- Confirm that the O-ring is not twisted.

6.2 Air filter

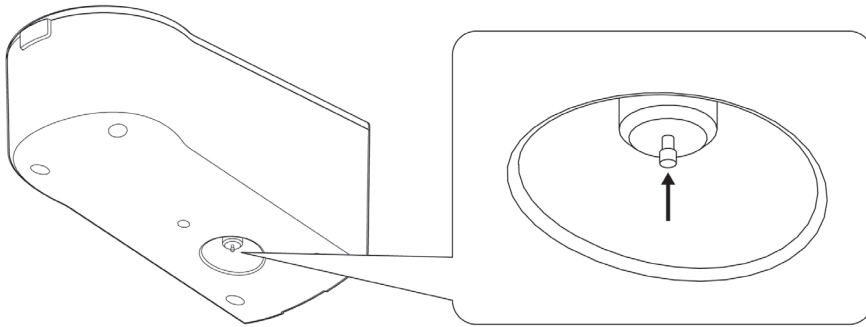1. Set the water bottle module supplied with air.
2. Push the end of the air filter to drain water.

6.3 Periodical maintenance

Refer to the check sheet below and perform periodical maintenance checks every month. Contact your authorized NSK dealer if any abnormalities are found.

Points to check	Details
Air filter	Drain water with air supplied to the water bottle module.

6.4 Replacing consumable parts

Parts name	Replacement time	Remarks	Replacing method
O-ring	3,000 uses or one year, whichever is sooner	For bottle lid	6.1.1 Irrigation bottle and CLM bottle
		Purge tool	6.1.2 Purge tool
Gasket	If there is water leakage or the gasket is damaged	For bottle lid	6.1.1 Irrigation bottle and CLM bottle
Irrigation bottle	6,000 uses or two years, whichever is sooner	-	-
CLM bottle	500 uses or two years, whichever is sooner	-	-

7 Troubleshooting

7.1 Problems and solutions

Problem	Cause	Remedy
Irrigation volume is low.	Clogged filter due to deterioration over time.	Air filter needs to be replaced. Contact your authorized NSK dealer.
	Bent or pinched water hose.	Refer to "3.2 Connecting hose" and attach the water hose properly.
Water leaking. (Bottle joint, bottom of the water bottle module, irrigation bottle, CLM bottle)	Worn O-ring or gasket of the bottle lid.	Replace the O-ring or gasket with a new one.
	Damaged bottle or bottle lid.	Replace the bottle or bottle lid with a new one.
Air leaking.	Damaged drive part inside the water bottle module.	Contact your authorized NSK dealer.
	Damaged air circuit part inside the water bottle module.	Contact your authorized NSK dealer.
	A supply air pressure outside of the specified value was used.	Confirm the supply pressure and adjust to the specified pressure. If air continues to leak after adjustment, contact your authorized NSK dealer.
The bottle cannot be removed.	The O-ring is worn.	Replace the O-ring.
	Damage inside the water bottle module.	Contact your authorized NSK dealer.


8 Spare Parts and Option List

Model	REF	Remarks
Irrigation Bottle Set	ZA16160001	
CLM Bottle Set	ZA16240001	
Bottle Lid	ZA16160100	
Gasket	ZA16160108	
O-ring	D031210010	
Irrigation Bottle	ZA16160112	Bottle only
CLM Bottle	ZA16240100	Bottle only
Water Hose	ZA16140121	0.7 m
Air Hose	ZA16140122	0.7 m
Air Hose	UA14210250	3.6 m
Purge Tool	ZA16300001	
b.clean	BCL600	Solution used in CLM mode

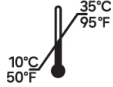9 Specifications

Model	Water Bottle Kit
Bottle maximum volume	950 mL
Supply air pressure	0.45-0.75 MPa (4.5-7.5 kgf/cm ²)
Supply water pressure	0.1-0.2 MPa
Dimensions	W217×D96×H274 mm (Including the irrigation bottle)
Weight	1.8 kg

■ Use, transportation and storage environment

	Temperature	Humidity	Atmospheric pressure
Use environment			-
Transportation and storage environment			

*No moisture condensation in the control unit.

*Use outside of these limits may cause malfunction.

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NSK Europe GmbH