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### 1.1 EN User and intended purpose

- User  
Personnel with knowledge and skills required for operating this product.
- Intended purpose  
The CLM module can simultaneously perform CLM mode for the water lines of two powder chambers and powder handpiece hoses of the Varios Combi Pro2 unit by connecting the devices (for decontamination and preventing the formation of biofilm).

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PRECAUTIONS

#### EN Restriction of use

- This product shall be used only in combination with Varios Combi Pro2, b.clean and Bilpron (ALPRO), never use with other devices.
- Be sure to use only the accessories of this product included in the package and NSK's genuine products (designated items). The user is liable for accidents and malfunctions that occur as a result of the use of items other than the accessories of this product included in the package and NSK's genuine products (designated items).

#### Precautions

- Do not drop, hit, or subject to excessive shock. Doing so could cause malfunctions.
- Before unpacking this product, make sure that the packaging has not been opened, tampered with, or damaged. If it has been opened or damaged, there may be a risk of malfunction or contamination. Stop use and contact your authorized NSK dealer.
- Use or store this product in a location that meets the requirements for the use environment and does not produce condensation. Otherwise, it may cause malfunction.
- Hold the plug when pulling out the handpiece hose. Pulling the hose may cause a broken wire or damage.
- If water droplets, chemicals, solvents, or liquid disinfectant come into contact with this product (including each part and each connection part), wipe them off immediately after use. Failure to do so may result in discoloration or deformation.
- This product is for indoor use only.
- Before connecting the products, make sure that there is no dirt or foreign objects adhered to each connection part. If there is any dirt or foreign objects, clean it off.
- One of this product can be connected to a single control unit. Do not connect more than one of this product to a control unit.
- Read this operation manual before use to fully understand the product functions and file for future reference.
- Do not attempt to disassemble the product or tamper with the mechanism except as recommended by NSK in this manual.

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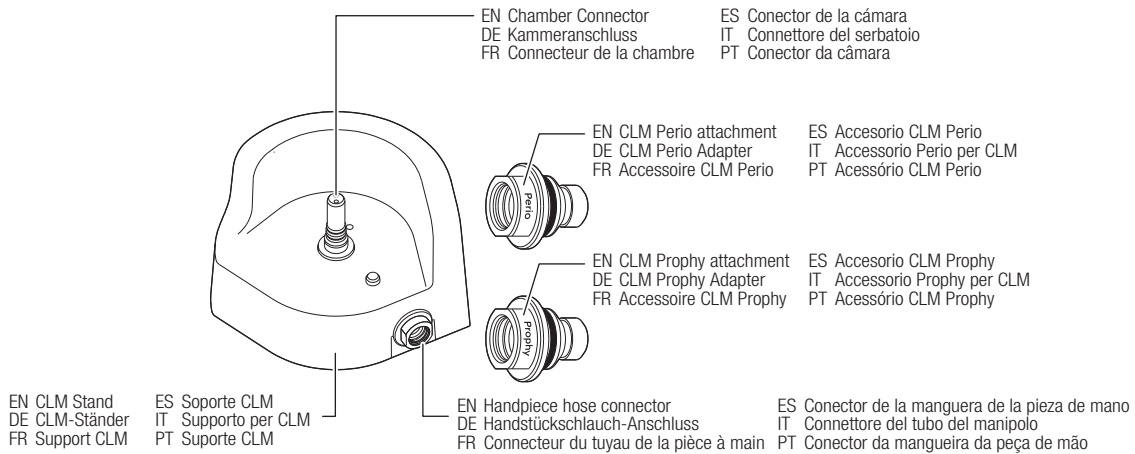
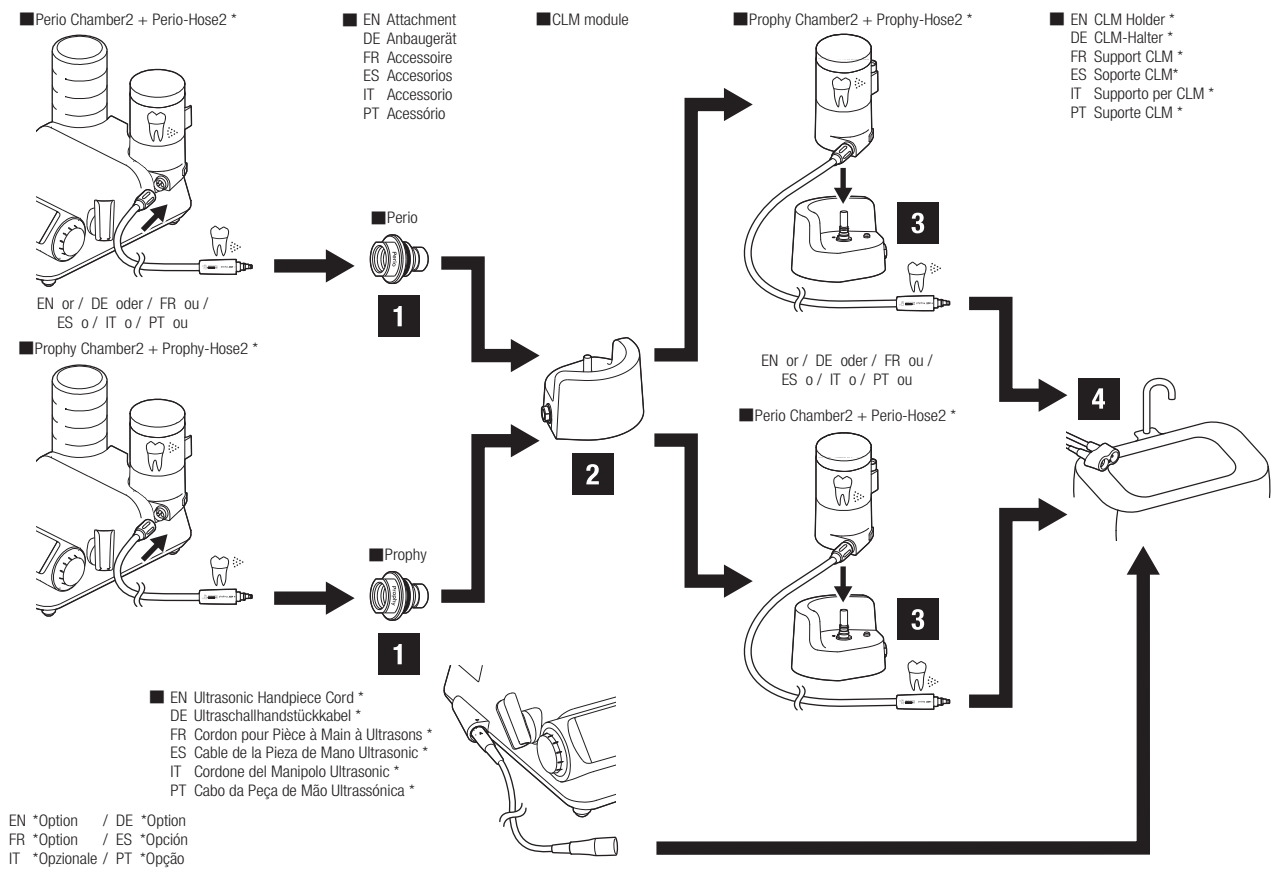
3.1 EN Connection to the chamber

- Select the CLM Perio/Prophy attachment relating to the chamber to be connected.
- Tighten the CLM Perio/Prophy attachment using the wrench ( **REF** K133751) delivered with the Varios Combi Pro2 accessory set.
- Make sure that there is no dirt or foreign objects adhered to each connection part.
- Insert the powder handpiece hose inside the CLM Perio/Prophy attachment
- Proper connection is confirmed by an audible “click”.

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
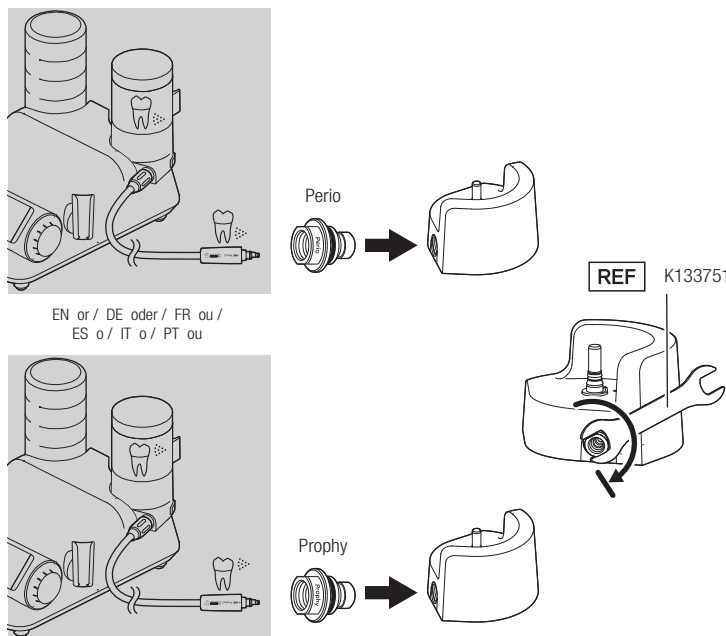
1

2
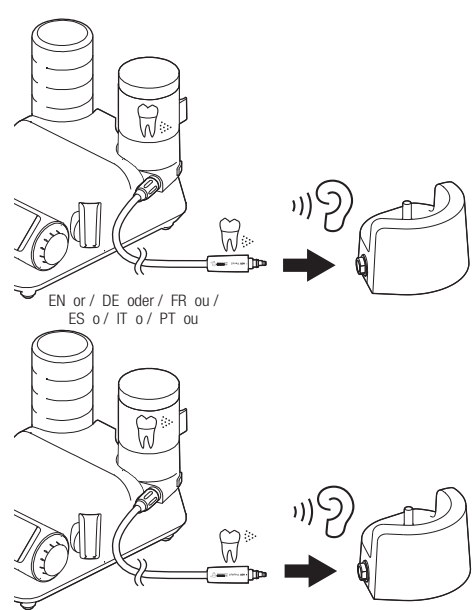
### 3.2 EN Installation of an additional chamber

- Simply insert the second chamber onto the CLM stand.
- Proper connection is confirmed by an audible "click".
- To achieve the installation, connect the ultrasonic handpiece cord handpiece to the CLM holder delivered with the Varios Combi Pro2.
- To remove, push the chamber lock button (A), and then pull out the chamber straight.

#### NOTICE

- Proceed to "Maintenance of water circuit (twice per week)".
- For water circuit maintenance, refer to "5.10 Decontamination and prevention of biofilm formation of water circuit (twice per week)" in OM-DE1002MA Varios Combi Pro2 OPERATION MANUAL.

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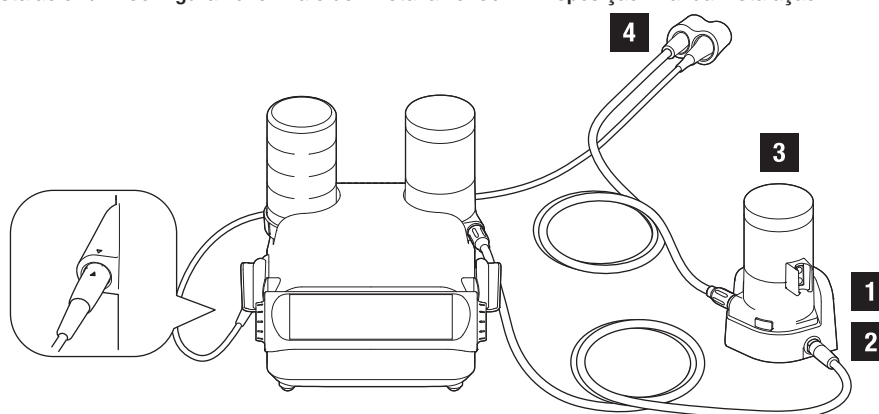
#### 4.1 EN Cleaning / Disinfection

- Clean the CLM stand and CLM Perio/Prophy attachment with lint-free cloth dampened with disinfectant alcohol.
- After cleaning, perform a visual inspection to ensure there are no residual soils.

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EN For cleaning and disinfection, use disinfectant ethanol or disinfectant isopropanol that does not contain any additives (any quaternary ammonium salts). Use of other disinfectants may cause discoloration or cracking.

\* For details, contact the manufacturer of the disinfectant.

\* In this operation manual, disinfectant ethanol or disinfectant isopropanol that does not contain any additives (any quaternary ammonium salts) is called by "disinfectant alcohol".

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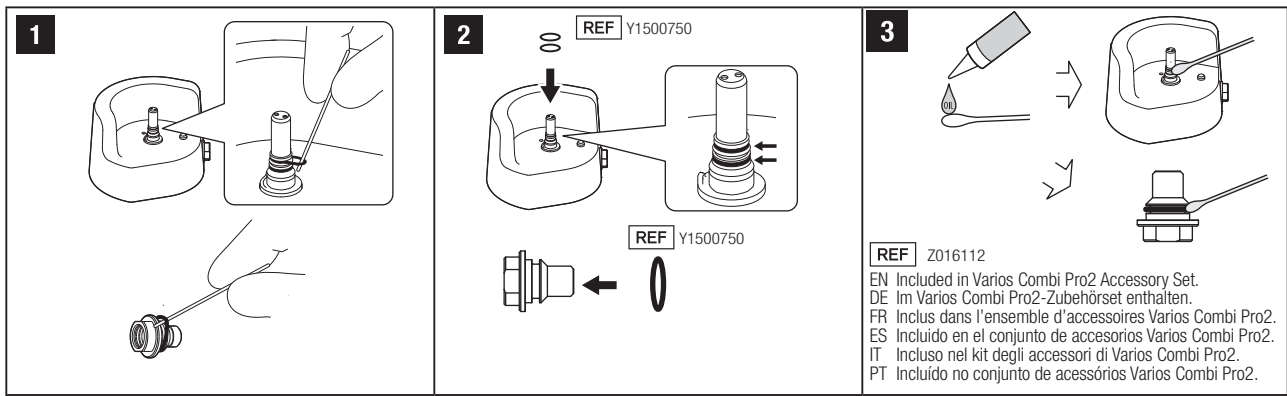
**5 EN Disposing of products**

When your medical device has reached the end of its service life, it must be disposed of via a specialist collection, removal, recycling or destruction channel. Prior to doing so, in order to avoid the health risks to operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, you need to confirm the equipment is decontaminated. Therefore contact your nearest NSK dealer, or one of the NSK branches to find out how to proceed. Refer to the link below:

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